

# S-CAPE PLUS


THE EVACUATION SOLUTION




MANUAL HANDLING | GEBRUIKSAANWIJZING  
GEBRAUCHSANLEITUNG | MODE D'EMPLOI  
INSTRUCCIONES DE USO | ISTRUZIONE PER L'USO  
KULLANIM KILZAVUZU | 活用マニュアル

Fast and safe evacuation of immobile people. Easily draggable by one rescuer.

Never overestimate yourself! An evacuation via the stairs can be very dangerous. Someone's entire weight will be coming down towards you and you must be sure you are capable of keeping control of this weight across the entire length of the stairs. If you are in any doubt whatsoever, ask a second rescuer for help, or leave it to others. Your own safety should always be your first priority. Practice regularly with the S-CAPE PLUS and always follow the instructions.

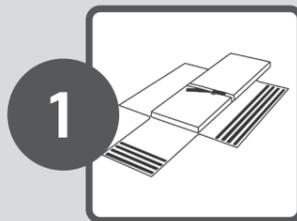

**1** Remove the S-CAPE PLUS from the bag. Fold out the flaps.

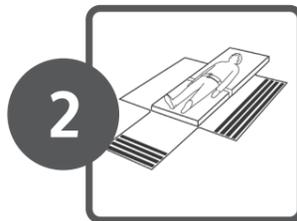

**2** Position the patient in the middle of the mattress. Make sure the positioning belt is placed right around the person's torso by pulling the belt through the ring on the right hand side. Fix this in place with the velcro strap. Position the arms along the body and above the belt.



**3** Now firmly pull the long flaps towards each other. Connect and stick the velcro strap to the right flap with the velcro strap on the left flap. Close this up as tightly as possible. Now close up the flap at the foot end. Again, do this as tightly as possible.



**4** Take a firm hold of the loop at the foot end, allowing you to evacuate the patient in a dragging motion. A second rescuer can use the loop at the head end to navigate more easily around bends.



**5** If you find yourself on a higher floor, you will need to use the stairs to make your way downstairs, as you won't be able to use any possible lift present during a fire. This will not present any problems, providing you take command of the stairs. Make sure you have enlisted the help of a second rescuer to hold the loop at the head end. This person can be responsible for any necessary braking when making your way down the stairs. Position yourself a few treads lower and pull the patient towards you. Carefully lower the mattress once the patient has crossed the stairwell with his/her buttocks. Make sure you keep a firm hold of the railing at all times and always maintain contact with the S-CAPE PLUS. Don't miss out any treads when going down the stairs and gradually do so step by step.

Only use in case of emergency situations. The manufacturer and supplier cannot be held liable for any improper use!

The fast and safe way to evacuate immobile people. Can be easily dragged by just one rescuer.

Never overestimate yourself! Evacuation via a staircase can be very dangerous! You will have to support a heavy weight and you must be certain that you are capable and strong enough to keep this weight under control all the way down the staircase. If in doubt as to whether you are able to evacuate somebody down a staircase yourself, you must always ask a second care worker to help you or leave the task to someone else. Your own safety is of the utmost importance! Regularly practicing using the S-CAPEPLUS during evacuation exercises and always observe the instructions given below.

## Safety information

These instructions contain information concerning the safe use of the S-CAPEPLUS. To ensure that the S-CAPEPLUS is used correctly and safely, this information must be read prior to placement and usage of the S-CAPEPLUS.

- Always observe the instructions given in the instruction manual when using the S-CAPEPLUS.
- Only use the S-CAPEPLUS in emergencies, such as a fire, power cut, gas leak, terrorist attack, natural disaster, extreme smoke or any other life-threatening situation or emergency.
- Product training with the S-CAPEPLUS must always be given by a certified emergency response team member.
- The use of alcohol and/or prescription and over-the-counter medicine may have a negative effect on the skill of the rescuer who pulls the S-CAPEPLUS.
- Do not remove any parts of the S-CAPEPLUS.
- Make sure the safety information and instruction manual are kept close to where the S-CAPEPLUS has been installed.
- This S-CAPEPLUS is not intended for recreational use.
- Do not use flammable vapours, materials or liquids, such as petrol, alcohol, etc., close to the S-CAPEPLUS.
- Not observing the instructions given in the instruction manual provided with the S-CAPEPLUS may result in serious physical injury or even death.
- Sharp edges or protrusions may damage the S-CAPEPLUS and, therefore, reduce the effectiveness of the product.
- Practice using the S-CAPEPLUS at least once a year in order to guarantee its correct and safe use in the event of an emergency.
- Contact the various rescue services (fire brigade, ambulance service, etc.) to inform them of the purchase and presence of the S-CAPEPLUS.
- The S-CAPEPLUS is not suitable to be pulled over a naked flame.
- The S-CAPEPLUS must be carefully checked for damage after it has been used. If damage is observed, the S-CAPEPLUS must not be reused. Furthermore, the S-CAPEPLUS must be folded up and stored away each time after use.
- The S-CAPEPLUS has a 1-year warranty for material and manufacture faults, which is valid from the date of purchase. The manufacturer may ask you for proof of the purchase date. Therefore, it is recommended to keep the invoice in a safe place. Faults or problems with the use of the S-CAPEPLUS as a result of an accident, misuse, incorrect use, alterations, vandalism, incorrect placement, incorrect maintenance or administration, as well as overdue normal and routine maintenance, are not covered by the warranty.
- The manufacturer is not liable for (consequential) damage as a result of the use of the S-CAPEPLUS.

Only use in case of emergency situations. The manufacturer and supplier cannot be held liable for any improper use!



**'Don't hesitate, use the S-CAPEPLUS  
to evacuate'**

**[s-capeplus.com](http://s-capeplus.com)**